



# BiopSafe®



Your daily protection.

BiopSafe® is a revolutionary method of handling biopsies which helps protect medical personnel all over the world. The big challenge in hospitals and clinics is how to fixate the biopsy in formalin without exposure to spillage or vapor.

BiopSafe® solves this problem - A small container with formalin encapsulated in the lid. With a light press of the thumb, the formalin is released in a closed system covering the biopsy. The result is that no personnel are exposed to the content in either liquid or vapor.

Features:

- Formalin encapsulated in the lid
- Easy press of the thumb to operate
- No exposure of personnel

The vials come in two sizes: 20 ml and 60 ml.

The 20 ml is perfect for easy and fast fixation of a single biopsy. The 60 ml makes it easy to handle more biopsies at the same time. Place each biopsy in a cassette, and arrange the cassettes in the vial. Screw on the and release the formalin.





**1.** Twist off the lid



**2.** Place the biopsy in the vial



**3.** Twist on the lid and tighten



**4.** A press of the thumb and the formalin is released



**5.** The biopsy is submerged in the formalin



**6.** The biopsy is ready for safe transportation

## Ordering Information

Ref no.	Country	Description	Qty/box
3178-200-08-FI	Finland	BiopSafe® - Biopsy Container, 20ml	192
3178-600-08-FI	Finland	BiopSafe® - Biopsy Container, 60ml	72
3178-200-21-NO	Norway	BiopSafe® - Biopsy Container, 20ml	192
3178-600-21-NO	Norway	BiopSafe® - Biopsy Container, 60ml	72
3178-200-18-SE	Sweden	BiopSafe® - Biopsy Container, 20ml	192
3178-600-18-SE	Sweden	BiopSafe® - Biopsy Container, 60ml	72
3178-200-25	UK	BiopSafe® - Biopsy Container, 20ml	192
3178-600-25	UK	BiopSafe® - Biopsy Container, 60ml	72
For safe handling and transportation • Plexitray for 20ml & 60ml BiopSafe			
VASSOFORM8		Plexitray - 8 containers (20 ml)	1
VASSOFORM12		Plexitray - 12 containers (20 ml)	1
VASSOFORM20		Plexitray - 20 containers (20 ml)	1
VASSOFORM60		Plexitray - 8 containers (60 ml)	1

### Contact Details, Finland, Norway & Sweden

Frydensbergvej 25 · DK-3660 Stenløse  
 customer@mermaidmedical.com  
 mermaidmedical.com

### Contact Details, United Kingdom

3A Cable Street, Formby · Merseyside L37 3LU  
 customer@mermaidmedical.co.uk  
 mermaidmedical.com